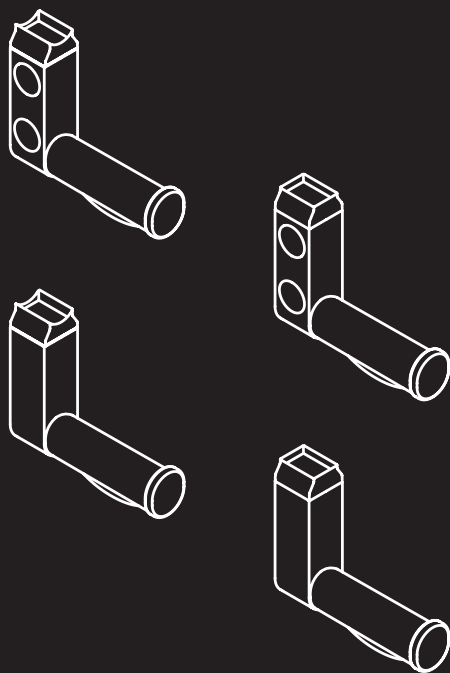
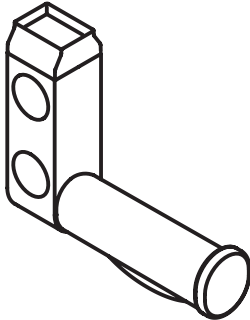
# HEWI

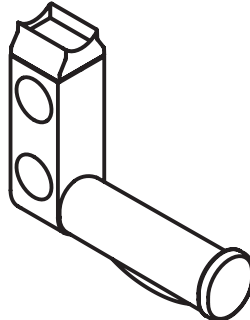
- Ⓓ Montage- und Gebrauchsanleitung
- ⒼⒷ Installation and Operating Instructions
- Ⓕ Notice de montage et mode d'emploi
- ⒼⓁ Installatie-en gebruikshandleiding
- Ⓘ Istruzioni di montaggio e per l'uso


950.50.06490  
950.50.06590


950.50.06290  
950.50.06390

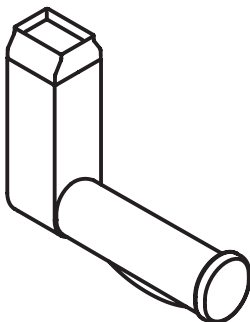950.50.01190  
950.50.01150

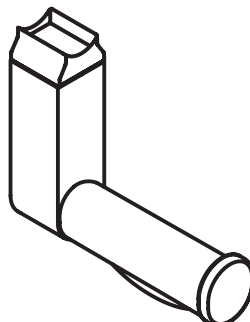950.50.01090  
950.50.01050

Ⓓ

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**GB**

**Mounting should be performed solely by qualified specialists.**

**Care and cleaning instructions**

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough. When using detergents, please observe the following:

- Only use detergents with a pH of between 6 and 8.
- Detergents containing active substances such as phosphates, soap and tensides may be used.
- Follow the detergent manufacturer's instructions (concentration, contact time).
- Detergents containing acids, alkalis, bleach or scouring agents may not be used.
- Do not use scouring cleaning equipment, such as brushes

**Exclusion of liability**

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of improper mounting or maintenance, particularly if the safety instructions and other points to note listed are not or have not been observed in full. HEWI accepts no liability whatsoever for damage occurring from incorrect cleaning.

**For further information, please contact HEWI Technical Support, tel. +44 1634 377688.**

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**Caution!**

The yellow button for triggering a „function“ may not be used as part of a call system to DIN VDE 0834-1.

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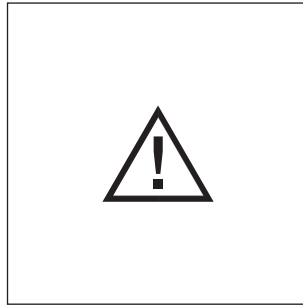
**10. Caution! Consult cistern manufacturer's instructions.**

Remove toilet flush operating panel prior to programming cistern control.

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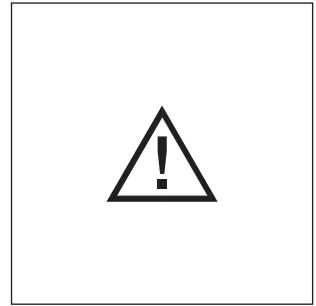
**11. Caution! Consult cistern manufacturer's instructions.**

Open cistern control lid. Cistern control can be programmed on left- and/or right-hand HEWI hinged support rail.

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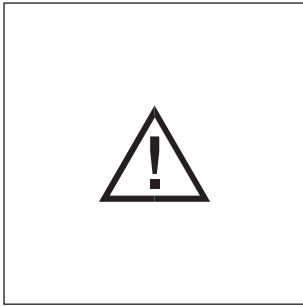
### 12. Caution! Consult cistern manufacturer's instructions

Deleting receiver settings: Prior to programming left- and/or right-hand HEWI hinged support rail via button on cistern control.

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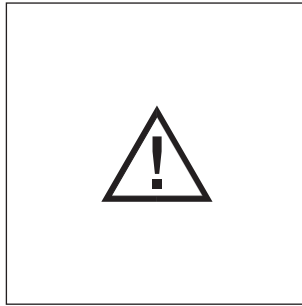
### 13. Caution! Consult cistern manufacturer's instructions.

Programming cistern control: Via button on cistern control and blue button (toilet flush) on HEWI hinged support rail. Same procedure for second HEWI hinged support rail.

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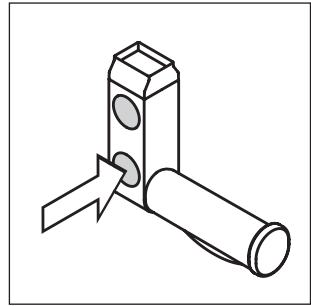
### 14. Test functionality by operating button.

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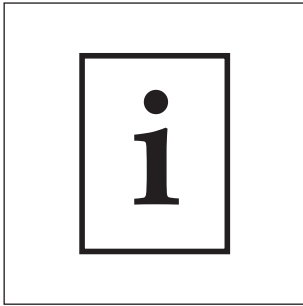
**Technical data**

Transmission frequency 868.4 MHz.  
Transmission output max. 5 mW.  
Operating voltage: 2 x 1.5 V mignon (AA).  
Allocate one cistern control for a  
maximum of 2 HEWI hinged support  
rails.

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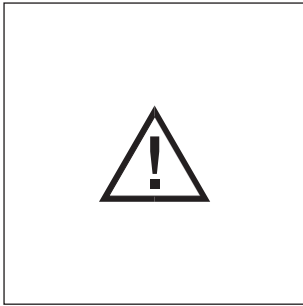
### **Important to note!**

Equipped with 2 batteries: type – 1.5 V, AA, LR6.  
Do not dispose of old batteries as standard waste! Always pull battery holder out of housing prior to dismantling.

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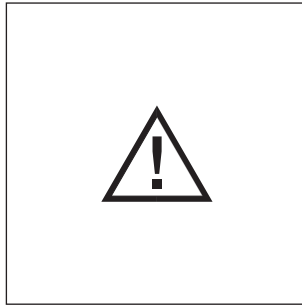
### **Measures to be taken in case of incorrect programming**

- Check power supply (correct connection and voltage of batteries)
- Delete receiver settings as per point 12.
- Programme cistern control on hinged support rail as per Point 13.

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HEWI Heinrich Wilke GmbH